ENGLISH

MONTHLY IMMUNOASSAY PROGRAMME: RQ9130

CONFIRMATION OF KIT CHARACTERISTICS AND RECEIPT DATE

Please confirm that the correct number of samples are present and that your samples have the appearance as indicated in the CHARACTERISTICS section below. Please confirm that none of the vials are broken and notify your local Randox representative immediately if there are any discrepancies. Finally, please log on to www.riqas.net to confirm the exact date on which you received this kit.

CHARACTERISTICS

The pack contains 6 vials of lyophilised 5 ml samples. The vials are labelled with the sample number.

PREPARATION

The samples are sealed under vacuum. Open the vial very carefully, avoiding any loss of material and using a **calibrated pipette** reconstitute with an accurately measured **5 ml volume** of freshly double distilled water at **+20°C to +25°C**. Replace the rubber stopper, close vial and leave to stand for **30 minutes** out of bright light before use. Ensure that the lyophilised contents are completely dissolved by swirling gently. Do not shake the vial. Analyse samples within 2 days of reconstitution in the absence of bacterial contamination with the following exceptions.

ACTH and Parathyroid hormone should be analysed immediately after reconstitution. Androstenedione and Thyroglobulin should be tested within 4 hours of reconstitution. C-Peptide should be analysed within 1 day of reconstitution. The samples should be treated in the same way as patient samples. The samples should be stored at 2 - 8°C when not in use.

SAFETY

The serum is human based. It has been source tested and found to be negative for HBsAg and antibodies to HIV and HCV. For complete protection however, it is recommended that the serum be handled as carefully as patient specimens.

For **IN VITRO** use only.

*** IMPORTANT NOTE:** Results must arrive at **RIQAS** by **17:00 hrs GMT** on the **FINAL DATE**. If the **RECOMMENDED ANALYSIS DATE** gives insufficient time, we suggest that the sample is analysed earlier to ensure you meet the deadline. If you are faxing results, please transmit **3 working days before the FINAL DATE**. Late results will not be accepted after the final date for the next sample.

MONTHLY IMMUNOASSAY PROGRAMME / PROGRAMME IMMUNOENZYMOLOGIE MENSUEL / PROGRAMMA MENSILE IMMUNOMETRIA /
 PROGRAMA DE INMUNOENSAYO MENSUAL / PROGRAMA DE IMUNOENSAIO MENSAL / IMMUNOASSAYPROGRAM
 IMMUNOASSYPROGRAM / 按月免疫分析方案 / CHƯƠNG TRÌNH NGOẠI KIỂM MIỄN DỊCH

RETURN OF RESULTS / DATES de RETOUR / INVIO DEI RISULTATI / ENVIO DE RESULTADOS
 PLAN FOR INSENDELSE AF / RESULTATRETUR/ ΚΥΚΛΟΣ ΕΠΙΣΤΡΟΦΗΣ / 回复结果 / GỬI TRẢ KẾT QUẢ

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