**ENGLISH**

**MATERNAL SCREENING: RQ9137**

**CONFIRMATION OF KIT CHARACTERISTICS AND RECEIPT DATE**

Please confirm that the correct number of samples are present and that your samples have the appearance as indicated in the CHARACTERISTICS section below. Please confirm that none of the vials are broken and notify your local Randox representative immediately if there are any discrepancies. Finally, please log on to [www.riqas.net](http://www.riqas.net) to confirm the exact date on which you received this kit.

**CHARACTERISTICS**

The pack contains 6 vials of lyophilised material (6 x 1 ml). The vials are labelled with the sample number.

**STORAGE**

The samples should be stored at 2-8°C.

**PREPARATION**

The samples are sealed under vacuum. Open the vial very carefully, avoiding any loss of material and using a **calibrated pipette** reconstitute with an accurately measured **1 ml volume** of freshly double distilled water at **+20°C to +25°C**. Replace the rubber stopper, close vial and leave to stand for **30 minutes out of bright light** before use. Ensure that the lyophilised contents are completely dissolved by swirling gently. Do not shake the vial. Once reconstituted, the samples should be analysed within 2 days. The samples should be treated in the same way as patient samples. The samples should be stored at 2 - 8°C when not in use.

**SAFETY**

The serum is human based. It has been source tested and found to be negative for HBsAg and antibodies to HIV. For complete protection however, it is recommended that the serum be handled as carefully as patient specimens.

For IN VITRO use only.

**\* IMPORTANT NOTE:** Results must arrive at **RIQAS** by **17:00 hrs GMT** on the **FINAL DATE**. If the **RECOMMENDED ANALYSIS DATE** gives insufficient time, we suggest that the sample is analysed earlier to ensure you meet the deadline. If you are faxing results, please transmit **3 working days before the FINAL DATE**. Late results will not be accepted after the final date for the next sample.

MATERNAL SCREENING PROGRAMME / PROGRAMME HORMONES DE GROSSESSE / PROGRAMMA SCREENING MATERNO /  
 PROGRAMA DE SCREENING MATERNAL / PROGRAM DESPISTE MATERNAL / NĚŠČIJ MOTERŮ STEBĚJIMO PROGRAMA /  
 โปรแกรม MATERNAL SCREENING / PROGRAM MATCZYNE TESTY PRZESIEWOWE / 孕妇筛查程序 / CNTK SÀNG LỌC TRƯỚC SINH

RETURN OF RESULTS / RETOUR DES RESULTATS / INVIO DEI RISULTATI/ ENVIO DE RESULTADOS /  
 REZULTATŮ GRAŽINIMAS / ตารางการส่งผล RIQAS กลับ / PRZESYŁANIE WYNIKÓW / 回复结果 / GỬI TRẢ KẾT QUẢ

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