ENGLISH

MONTHLY HAEMATOLOGY PROGRAMME: RQ9140A

CONFIRMATION OF KIT CHARACTERISTICS AND RECEIPT DATE

Please confirm that the correct number of samples are present and that your samples have the appearance as indicated in the CHARACTERISTICS section below. Please confirm that none of the vials are broken and notify your local Randox representative immediately if there are any discrepancies. Finally, please log on to www.riqas.net to confirm the exact date on which you received this kit.

Important Notice: The Haematology control material is not suitable for use on the QBC Autoread Plus Analyser

CHARACTERISTICS

12 samples are supplied per cycle and are shipped in packs of three. This pack contains samples 1-3. Each pack contains 3 x 2 ml primary tubes of human-based, preserved, whole blood samples.

PREPARATION

- 1) Remove the sample tube to be analysed from the refrigerator and allow the sample to warm at room temperature for 15 minutes.
- 2) Mix the sample by holding the tube horizontally between the palms of the hands. Do not pre-mix on a mechanical mixer. Incomplete mixing of the tube prior to analysis invalidates the sample withdrawn and any remaining material in the tube.
 - a) Roll the tube back and forth for 20-30 seconds, occasionally inverting the tube. Mix vigorously but **DO NOT SHAKE**.
 - b) Continue to mix in this manner until the red cells are completely suspended.
 - c) Gently invert the tube 8-10 times immediately before analysing the sample.
- 3) After sampling:
 - a) Automatic sample handling: Remove the tube from the sample handler immediately after sampling.
 - b) Manual sample handling: Carefully wipe the rim of the tube and cap with a lint-free tissue and replace the cap.
- 4) Return the tube to the refrigerator within 30 minutes of use.

The samples should be treated in the same way as patient samples.

On the Siemens Advia 120/2120 the BASO White Blood Cell Count (CBC) should be reported.

On the Sysmex XN analyser, the value submitted for White Blood Cells (WBC) should be the Total Nucleated Cell (TNC) value (Nucleated Red Blood Cells (NRBC) plus WBC). This is only applicable where NRBC values are measured.

STABILITY

Samples should always be stored upright at 2 - 8°C when not in use. Do not allow tubes to overheat or freeze. Once opened please analyse the samples within 2 days.

SAFETY

Potentially hazardous biomaterial. Do not ingest. Each donor is tested by FDA approved methods for HbsAg, anti-HCV, HIV-1, HCV (RNA) and HIV-1/2 and found to be negative. Each donor is also found negative by a serological test for Syphilis (RPR or STS). However, it is recommended that the samples are considered to be potentially infectious and should be handled and disposed of in the same manner as patient samples.

For IN VITRO use only.

*** IMPORTANT NOTE:** Results must arrive at **RIQAS** by **17:00 HRS GMT** on the **FINAL DATE**. If the **RECOMMENDED ANALYSIS DATE** gives insufficient time, we suggest that the sample is analysed earlier to ensure you meet the deadline. If you are faxing results, please transmit **3 working days before the FINAL DATE**. **Late results will not be accepted after the final date for the next sample.**

MONTHLY HAEMATOLOGY PROGRAMME / PROGRAMME HEMATOLOGIE MENSUAL / PROGRAMMA DI EMATOLOGIA MENSILE /
 PROGRAMA MENSUAL DE HEMATOLOGIA / PROGRAMA DE HEMATOLOGIA MENSAL / ΜΕΝΕΣΙΝΕ ΗΜΑΤΟΛΟΓΙΝΕ ΠΡΟΓΡΑΜΜΑ /
 ΜΗΝΙΑΙΟ ΑΙΜΑΤΟΛΟΓΙΚΟ ΠΡΟΓΡΑΜΜΑ / MIESIECZNY PROGRAM HEMATOLOGICZNY / 血常规项目 / CHƯƠNG TRÌNH NGOẠI KIỂM HUYẾT HỌC

RETURN OF RESULTS / RETOUR DES RESULTATS / RITORNO DEI RISULTATI / ENTREGA DE RESULTADOS /
 ENVIO DE RESULTADOS / CYKEL FÖR / KYKAOS EΠΙΣΤΡΟΦΗΣ / 返回结果 / GỬI TRẢ KẾT QUẢ

CYCLE 12 / CICLO 12 / CYCLE 12 / CICLO 12 / CICLO 12 / CYCLE 12 / CYKL 12 / 循环 12 / CHU KỶ 12

SAMPLE NO / ECHANTILLON NO. /
 N. CAMPIONE / N° DE MUESTRA /
 AMOSTRA N°. / MÉGINIO NR. /
 ΑΡ. ΔΕΙΓΜΑΤΟΣ /
 NR PRÓBKÍ / 样品号 / MẪU SỐ

RECOMMENDED ANALYSIS DATE/ DATE RECOMMANDE POUR L' ANALYSE/
 DATA DI ANALISI RACCOMANDATA/ FECHA DE ANALISIS RECOMENDADA/
 DATA RECOMENDADA PARA ANÁLISE/ REKOMENDUOJAMA MÉGINIO TYRIMO DATA/
 ΣΥΝΙΣΤΩΜΕΝΗ ΗΜΕΡΟΜΗΝΙΑ ΑΝΑΛΥΣΗΣ/
 ZALECANA DATA WYKONANIA OZNACZENIA/推荐分析日期/NGÀY KHUYẾN CÁO PHẦN TÍCH

* FINAL DATE/ *DATE LIMITE/
 * ULTIMO GIOMO/ *FECHA FINAL/
 * DATA FINAL/ *PASKUTINĖ DATA/
 * ΤΕΛΗ ΗΜΕΡΟΜΗΝΙΑ/
 * DATA KOŃCOWA/ *最终日期/
 * HẠN CUỐI GỬI TRẢ KẾT QUẢ

SAMPLE NO / ECHANTILLON NO. / N. CAMPIONE / N° DE MUESTRA / AMOSTRA N°. / MÉGINIO NR. / ΑΡ. ΔΕΙΓΜΑΤΟΣ / NR PRÓBKÍ / 样品号 / MẪU SỐ	RECOMMENDED ANALYSIS DATE/ DATE RECOMMANDE POUR L' ANALYSE/ DATA DI ANALISI RACCOMANDATA/ FECHA DE ANALISIS RECOMENDADA/ DATA RECOMENDADA PARA ANÁLISE/ REKOMENDUOJAMA MÉGINIO TYRIMO DATA/ ΣΥΝΙΣΤΩΜΕΝΗ ΗΜΕΡΟΜΗΝΙΑ ΑΝΑΛΥΣΗΣ/ ZALECANA DATA WYKONANIA OZNACZENIA/推荐分析日期/NGÀY KHUYẾN CÁO PHẦN TÍCH	* FINAL DATE/ *DATE LIMITE/ * ULTIMO GIOMO/ *FECHA FINAL/ * DATA FINAL/ *PASKUTINĖ DATA/ * ΤΕΛΗ ΗΜΕΡΟΜΗΝΙΑ/ * DATA KOŃCOWA/ *最终日期/ * HẠN CUỐI GỬI TRẢ KẾT QUẢ
1	07.01.19	14.01.19
2	04.02.19	11.02.19
3	04.03.19	11.03.19

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