**English**

**SPECIFIC PROTEINS PROGRAMME: RQ9160**

**CONFIRMATION OF KIT CHARACTERISTICS AND RECEIPT DATE**

Please confirm that the correct number of samples are present and that your samples have the appearance as indicated in the CHARACTERISTICS section below. Please confirm that none of the vials are broken and notify your local Randox representative immediately if there are any discrepancies. Finally, please log on to [www.riqas.net](http://www.riqas.net) to confirm the exact date on which you received this kit.

**CHARACTERISTICS**

The pack contains 12 vials of liquid 2 ml samples. The vials are labelled with the sample number.

**PREPARATION**

The samples are ready for use. To avoid evaporation, replace the rubber stopper and close vial immediately after use and store at +2 to +8°C. Once opened please analyse the sample within 2 days. The samples should be treated in the same way as patient samples. The samples should be stored at 2-8°C when not in use.

**SAFETY**

The serum is human based. It has been source tested and found to be negative for HBsAg and antibodies to HIV. For complete protection however, it is recommended that the serum be handled as carefully as patient specimens.

For IN VITRO use only.

**\* IMPORTANT NOTE:** Results must arrive at **RIQAS** by **17:00 HRS GMT** on the **FINAL DATE**. If the **RECOMMENDED ANALYSIS DATE** gives insufficient time, we suggest that the sample is analysed earlier to ensure you meet the deadline. If you are faxing results, please transmit 3 working days before the **FINAL DATE**. Late results will not be accepted after the final date for the next sample.

SPECIFIC PROTEINS PROGRAMME / PROGRAMME PROTEINES SPECIFIQUES / PROGRAMMA PER PROTEINE SPECIFICHE  
 PROGRAMA DE PROTEINAS ESPECIFICAS / PROGRAMA DE PROTEÍNAS ESPECÍFICAS / PROGRAM FOR SPECIFIKKE PROTEINER /  
 PROGRAM SPECIFIKA PROTEINER / ΠΡΟΓΡΑΜΜΑ ΕΙΔΙΚΩΝ ΠΡΩΤΕΪΝΩΝ / PROGRAM DO OZNACZANIA BIAŁEK SWOISTYCH / 特异性蛋白分析程序  
 CHƯƠNG TRÌNH NGOẠI KIỂM PROTEIN ĐẶC HIỆU

RETURN OF RESULTS / DATES de RETOUR / INVIO DEI RISULTATI / ENVIO DE RESULTADOS  
 PLAN FOR INSENDELSE AF / CYKEL FÖR / ΚΥΚΛΟΣ ΕΠΙΣΤΡΟΦΗΣ / GỬI TRẢ KẾT QUẢ

CYCLE 60 / CICLO 60 / RESULTATER 60 / RESULTATRETUR 60 /  
 ΑΠΟΤΕΛΕΣΜΑΤΩΝ 60 / CYKL ZWROTU WYNIKÓW 60 / 返回结果周期 60 / CHU KỶ 60

SAMPLE NO / ECHANTILLON NO. / FINALE / N. CAMPIONE / N° DE MUESTRA / SLUTDATO / AMOSTRA N°. / PRØVENR. / ΗΜΕΡΟΜΗΝΙΑ / ΑΡ. ΔΕΙΓΜΑΤΟΣ / PROV NR. / NR PRÓBKİ / 样品号 / MÃU SỐ	RECOMMENDED ANALYSIS DATE/ DATE RECOMMANDE POUR L' ANALYSE/ DATA DI ANALISI RACCOMANDATA/ FECHA DE ANALISIS RECOMENDADA/ DATA RECOMENDADA PARA ANÁLISE/ ANBEFALET ANALYSEDATO/ REKOMMENDERAT ANALYSDATUM/ ΣΥΝΙΣΤΟΜΕΝΗ ΗΜΕΡΟΜΗΝΙΑ ΑΝΑΛΥΣΗΣ ZALECANA DATA WYKONANIA OZNACZENIA/ 推荐分析日期/ NGÀY KHUYẾN CÁO PHẦN TÍCH	* FINAL DATE/ *DATE LIMITE/ DATA * FECHA FINAL/ DATA FINAL/ * * SLUTDATUM/ *TEΛΙΚΗ * DATA KOŃCOWA/ *最终日期 * HẠN CUỐI GỬI TRẢ KẾT QUẢ
--	---	---

1	18.03.19	25.03.19
2	25.03.19	01.04.19
3	08.04.19	15.04.19
4	22.04.19	29.04.19
5	06.05.19	13.05.19
6	20.05.19	27.05.19
7	03.06.19	10.06.19
8	17.06.19	24.06.19
9	01.07.19	08.07.19
10	15.07.19	22.07.19
11	29.07.19	05.08.19
12	12.08.19	19.08.19

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