ENGLISH
PRE-ECLAMPSIA PROGRAMME: RQ9207

CONFIRMATION OF KIT CHARACTERISTICS AND RECEIPT DATE

Please confirm that the correct number of samples are present and that your samples have the appearance as indicated in the CHARACTERISTICS section below. Please confirm that none of the vials are broken and notify your local Randox representative immediately if there are any discrepancies. Finally, please log on to www.riqas.net to confirm the exact date on which you received this kit.

CHARACTERISTICS

This pack contains 6 vials of lyophilised sample (6 x 1 ml). The vials are labelled with the sample number.

PREPARATION/STORAGE/STABILITY OF SAMPLES

The samples are sealed under vacuum. Open the vial very carefully, avoiding any loss of material and using a **calibrated pipette** reconstitute with an accurately measured **1 ml volume** of freshly double distilled water at **+20°C to +25°C**. Replace the rubber stopper, close vial and leave to stand for **30 minutes** before use. Ensure that the lyophilised contents are completely dissolved by swirling gently. Do not shake the vial. Analyse samples within 2 days of reconstitution in the absence of bacterial contamination.

The samples should be treated in the same way as patient samples. The samples should be stored at 2 - 8°C when not in use.

SAFETY

Biological source material. Treat as potentially infectious.

Each human donor unit used in the preparation of this product has been tested and found to be nonreactive for Hepatitis B Surface Antigen, antibodies to Human Immunodeficiency Virus (HIV) Types 1 and 2 and antibodies to Hepatitis C Virus. However, no known test method can assure that products derived from human sources will not transmit infection. It is recommended that this product and all human specimens be handled in accordance with Biosafety Level 2 practices as described in the World Health Organization Laboratory Biosafety Manual, or other equivalent guidelines.

Samples contain ProClin 300: Skin Sens. 1:H317 - May cause an allergic skin reaction. Please see SDS for more information (<https://inserts.randox.com/out.php>).

For **IN VITRO** use only.

*** IMPORTANT NOTE:** Results must arrive at RIQAS by **17:00 HRS GMT** on the **FINAL DATE**. If the **RECOMMENDED ANALYSIS DATE** gives insufficient time, we suggest that the sample is analysed earlier to ensure you meet the deadline. Late results will not be accepted after the final date for the next sample.

PRE-ECLAMPSIA PILOT STUDY / PRE-ECLAMPSIE (ETUDE PILOTE) / PRE-ECLAMPSIA (STUDIO PILOTA) /
ESTUDIO PILOTO PRE-ECLAMPSIA / PRÉ-ECLÂMPsia (ESTUDO PILOTO) / PRE-EKLAMPSJA (BADANIE PILOTAŻOWE) /
先兆子癇 (试点计划) / TIỀN SẢN GIẬT (NGHIÊN CỨU THÍ ĐIỂM) / PREEKLAMPSI PILOT ÇALIŞMASI /
PRE-ECLAMPSIA (PILOT STUDY)

RETURN OF RESULTS / RETOUR DES RESULTATS / INVIO DEI RISULTATI/ ENVIO DE RESULTADOS /
PRZESYŁANIE WYNIKÓW / 回复结果 / GỬI TRẢ KẾT QUẢ / SONUÇ GÖNDERİM TARİHLERİ / ODOŚLANIE WYŚLEDKÓV

CYCLE 2A / CICLO 2A / CIKLAS 2A / CYKL 2A / 返回结果周期 2A / CHU KỲ 2A / DÖNEM 2A / CYKLUS 2A

SAMPLE NO / ECHANTILLON NO. / N. CAMPIONE / N° DE MUESTRA / AMOSTRA N°. / NR PRÓBK / 样品号 / MẪU SỐ / ÖRNEK NO / Č.VZORKY	RECOMMENDED ANALYSIS DATE/ DATE RECOMMANDE POUR L' ANALYSE/ DATA DI ANALISI RACCOMANDATA/ FECHA DE ANALISIS RECOMENDADA/ ZALECANA DATA WYKONANIA OZNACZENIA/ 推荐分析日期 NGÀY KHUYẾN CÁO PHÂN TÍCH / ÖNERİLEN ANALİZ TARİHİ / ODPORÚČANÝ DÁTUM ANALÝZY	* FINAL DATE/ *DATE LIMITE/ * DATA FINALE/ *FECHA FINAL/ * DATA KOŃCOWA/ *最终日期 * HẠN CUỐI GỬI TRẢ KẾT QUẢ/ * SON SONUÇ GÖNDERİM TARİHİ/ * FINÁLNÝ DÁTUM
1	05 Jan 26	12 Jan 26
2	02 Feb 26	09 Feb 26
3	02 Mar 26	09 Mar 26
4	06 Apr 26	13 Apr 26
5	04 May 26	11 May 26
6	01 Jun 26	08 Jun 26

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